



# HPMed

Cleaner for hard surfaces

**HPMed** acts as an adjuvant to the sanitising action of " Polti Sani System ".

The **over-heated dry saturated steam** of "Polti Sani System" becomes the ideal vehicle for carrying the **temperature** and the **HPMed** sanitising agent. In this way, steam is able to reach even the least accessible corners.

HPMed also eliminates smells.

HPMed was subjected to skin tests and can be used in the presence of people.

**Instructions:** Open and fit onto the delivery nozzle the disposable bottle through the disposable adaptor, and switch on the "Sani System Polti" appliance following the manufacturer's instructions. Once the system is ready, distribute the steam and HPMed sanitising agent. Mixed with the steam, the product will dry within 30 seconds up to one minute, depending on the environmental conditions.

**Declaration of cleaning ingredients according to regulations  
(CEE N° 648/2004):**

Water (CAS 7732-18-5)

Ethilic alcohol (CAS 64-17-5 EICNECS 200-578-6)

Sodium metasilicate (CAS 6834-92-0 EICNECS 229-912-0)

Sodium carbonate (CAS 497-19-8 EICNECS 207-838-8)

Ethoxylated alcohol (CAS 34398-01-1)

Biodegradable; Contains <5% non-ionic surfactant (CEE Reg. 648/2004).  
Keep out of the reach of children. Do not ingest. In case of contact with the eyes, rinse thoroughly with water and consult a doctor.



**MEDICAL DIVISION**

Politi SpA – Capitale sociale Euro 6.000.000 – CF e P: IVA 01457440137 – N. Iscr. R.E.A. 191547 – Reg. Imp. n. CO 075-16547

Sede Legale, Uffici, Stabilimento Produttivo: 22070 Bulgarograsso (CO) – Italia – Via Ferloni 83

Medical Division – Uffici: 22077 Olgiate Comasco (CO) – Italia – Via Torino, 1/3 – Tel. 031.943.111 – Fax 031.943.455

[www.sanisystempoliti.com](http://www.sanisystempoliti.com)